

“Swivel Right Pro” User Instructions

- **Setup & User instructions:**

- 1. Hold the spoon handle in your dominant hand.
 - If the spoon head is not attached to the handle, place/gently push the black open ring of the swivel spoon head around the small cylinder that protrudes from the handle until it reaches the thicker cylinder shaft of the handle.
- 2. Place and twist the white dome on the end of the small cylinder that protrudes from the handle until it feels secure.
 - This will help keep the spoon head from falling off the spoon.
- 3. Continue to hold the spoon by the handle in your dominant hand as you reach to scoop whatever solid/liquid you plan to consume.
- 4. Bring the solid/liquid to your mouth.
 - You will notice the spoon head will swivel/move as you bring it to your mouth. This is okay! It helps keep the spoon and its contents upright until they make it to your mouth.
- 5. Open your mouth to receive the solid/liquid and enjoy!



- **Precautions:**

- When using the “Swivel Right Pro”, be careful not to swing the spoon with heavy force, this may cause the spoon to hit your face or to fling the food off of your spoon.
 - This also may cause the spoon to break.
- When attaching the spoon head to the handle, only use gentle force. Jamming the spoon head into the handle could cause the spoon head to break.
- It is essential to secure the spoon head to the handle by attaching the white dome mentioned in the user setup/instructions. If this step is skipped, the spoon head may fall off the handle, causing a break to the spoon or a spill of the food.
- This spoon is hand-wash only, do not place it in a dishwasher.
 - Additionally, hand wash with lukewarm or cold water to ensure that neither the handle nor the black material attaching the spoon head to the handle lose their shape.

- **Safety warnings:**

- DO NOT put the spoon in a high-heat environment (microwave or oven).
- Do not place the black material extending off of the spoon head in your mouth or ingest this material. This material can be toxic.

“Swivel Right Express” User Instructions

- **Setup & User instructions:**

- 1. Hold the manufactured swivel spoon in your non-dominant hand and the white handle attachment in your dominant hand. Position them to create either a “T” or an “L” shape when put together.
 - The rectangular cutout in the handle attachment should face you, such that you can see through the hole.
- 2. Take an attached hook and loop strand of velcro and feed it through the hole in the handle attachment mentioned above. Secure the velcro tightly so that the swivel spoon is attached to the handle attachment and does not move.
 - Repeat with a second strand of velcro if you feel that the handle needs more support.
- 3. Switch your grasp to hold the spoon by the handle in your dominant hand as you reach to scoop whatever solid/liquid you plan to consume.
- 4. Bring the solid/liquid to your mouth.
 - You will notice the spoon head will swivel/move as you bring it to your mouth. This is okay! It helps keep the spoon and its contents upright until they make it to your mouth.
- 5. Open your mouth to take in the solid/liquid and enjoy!



- **Precautions:**

- When using the “Swivel Right Express”, be careful not to swing the spoon with heavy force, this may cause the spoon to hit your face or to fling the food off of your spoon.
 - This also may cause the spoon handle attachment to release.
- When using the “Swivel Right Express”, make sure to secure velcro tightly around both pieces
 - Weakly secured velcro may lead to an unstable spoon.
- This spoon and handle attachment are hand-wash only, do not place it in a dishwasher.
 - Additionally, do not wet the velcro. Remove any velcro pieces for washing.

- **Safety warnings:**

- DO NOT expose the spoon to any high-heat environment (microwave or oven)
- Do not ingest velcro material.